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Certificate Regarding Air Transportation by Passengers with Walking Disabilities

We hereby certify that the product:

Heartway BRIO S19 mobility scooter

With Lithium Ion battery (77172679 | HT-18A):

25.2V / 10.05Ah / 253.2Wh

Of type:

UN 3481, P.I. 966 Section II

(Meets the requirements of UN manual of Tests and Criteria, Part III subsection 38.3)

May be carried-on passenger aircraft without restriction according to **IATA 2.3.2.4.**

It is strongly recommended that you make advanced arrangement with each airline operation and receive approval prior to departure date.

This certificate does not apply to damaged or defective batteries.

Heartway Medical Products Co., Ltd.

April 1, 2025

No.13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C.
TEL: 886-2-2601-0385 FAX: 886-2-2601-0460 <http://www.metco.com.tw>

SAFETY DATA SHEET

MANUFACTURER INFORMATION

Mobile Energy Technology Co., Ltd.
No.13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C.
TEL: 886-2-2601-0385 FAX: 886-2-2601-0460 <http://www.metco.com.tw>
Date: March 25, 2022 Version: A.0

1. PRODUCT INFORMATION

Product Name: Lithium Ion Rechargeable Battery Pack
Model No.: **HT-18A**
Cell: NCR18650GA
Rating: 25.2V/10.05Ah/253.26Wh
IATA Battery type code:
UN 3480 - Lithium Ion Batteries
UN 3481 - Lithium Ion Batteries and Lithium Ion batteries contained in equipment or packed with equipment.

2. HAZARDS IDENTIFICATION

For the battery cell, chemical materials are stored in a hermetically sealed metal or metal laminated plastic case, designed to withstand temperatures and pressures encountered during normal use. As a result, during normal use, there are no physical hazards such as ignition, explosion and chemical hazards due to leakage of battery contents.

However, if exposed to a fire, added mechanical shocks, decomposed, added electric stress by miss-use, the gas release vent will be operated. The battery cell case will be breached at the extreme, hazardous materials may be released.

Also, if it is heated strongly by surrounding fires or the like, there is a possibility that irritating or harmful gas may be generated.

• GHS classification: Not available
(This product is outside the scope of GHS system since it's considered as an "article".)

• Most important hazard and effects

Human health effects:

Inhalation: The steam of the electrolyte has an anesthesia action and stimulates a respiratory tract.

Skin contact: The steam of the electrolyte stimulates a skin. The electrolyte skin contact causes a sore and stimulation on the skin.

Eye contact: The steam of the electrolyte stimulates eyes. The electrolyte eye contact causes a sore and stimulation on the eye. Especially, substance that causes a strong inflammation of the eyes is contained.

Environmental effects: Since a battery cell remains in the environment, do not throw out it into the environment.

• Specific hazards:

If the electrolyte contacts with water, it will generate detrimental hydrogen fluoride.
Since the leaked electrolyte is inflammable liquid, do not bring close to fire.

No.13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C.
 TEL: 886-2-2601-0385 FAX: 886-2-2601-0460 <http://www.metco.com.tw>

3. COMPOSITION / INFORMATION ON INGREDIENTS

- Substance or preparation: Preparation
- Information about the chemical nature of product: *1

Portion	Material name	CAS No.	Concentration range (wt %)
Positive electrode	Lithium transition metal oxidate (Li[M] _m [O] _n ^b)	12190-79-3	20-60
		12031-65-1	
		12057-17-9	
		182442-95-1	
		207803-51-8	
Positive electrode's base	Aluminum	7429-90-5	1-10
Negative electrode	Carbon	7782-42-5	10-30
		7440-44-0	
Negative electrode's base	Copper	7440-50-8	1-15
Electrolyte	Ethyl methyl carbonate Diethyl carbonate Ethylene carbonate Lithium hexafluorophosphate	623-53-0	5-25
		105-58-8	
		96-49-1	
		21324-40-3	
Outer case	Aluminum, iron, aluminum laminated plastic	7429-90-5	1-30
		7439-89-6	

*a Not every product includes all of these materials.

*b The letter M means transition metal and candidates of M are Co, Mn, Ni and Al. One compound includes one or more of these metals and one product includes one or more of the compounds. The letter m and n means the number of atoms.

4. FIRST-AID MEASURES

Spilled internal cell materials

- Inhalation:

Make the victim blow his/her nose, gargle. Seek medical attention if necessary.

- Skin contact:

Remove contaminated clothes and shoes immediately. Wash extraneous matter or contact region with soap and plenty of water immediately.

- Eye contact:

Do not rub one's eyes. Immediately flush eyes with water continuously for at least 15 minutes. Seek medical attention immediately.

A battery cell and spilled internal cell materials

- Ingestion:

Wash out mouth thoroughly. Do not make the victim vomit, unless instructed by medical personnel. Seek medical attention immediately.

5. FIRE-FIGHTING MEASURE

- Suitable extinguishing media: Plenty of water, carbon dioxide gas, nitrogen gas, chemical powder fire extinguishing medium and fire foam.
- Specific hazards: Corrosive gas may be emitted during fire.
- Specific methods of fire-fighting: When the battery burns with other combustibles simultaneously, take fire-extinguishing method which correspond to the combustibles. Extinguish a fire from the windward as much as possible.

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No.13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C.

TEL: 886-2-2601-0385 FAX: 886-2-2601-0460 <http://www.metco.com.tw>

- Special protective equipment for firefighters: Refer to Section 8-EXPOSURE CONTROLS/PERSONAL PROTECTION (WHEN THE ELECTROLYTE LEAKS)

6. ACCIDENTAL RELEASE MEASURES

Spilled internal cell materials, such as electrolyte leaked from a battery cell, are carefully dealt with according to the followings.

- Precautions for human body:
Remove spilled materials with protective equipment (refer to Section 8-EXPOSURE CONTROLS/PERSONAL PROTECTION (WHEN THE ELECTROLYTE LEAKS)). Do not inhale the gas as much as possible. Moreover, avoid touching with as much as possible.
- Environmental precautions: Do not throw out into the environment.
- Method of cleaning up: The spilled solids are put into a container. The leaked place is wiped off with dry cloth.
- Prevention of secondary hazards: Avoid re-scattering. Do not bring the collected materials close to fire.

7. HANDLING AND STORAGE

- Handling suggestions
 - Do not connect the positive terminal to the negative terminal with electrical wire or chain.
 - Avoid polarity reverse connection when installing the battery to an instrument.
 - Do not wet the battery with water, seawater, drink or acid; or expose to strong oxidizer.
 - Do not damage or remove the external tube.
 - Keep the battery away from heat and fire.
 - Do not disassemble or reconstruct the battery; or solder the battery directly.
 - Do not give a mechanical shock or deform.
 - Do not use unauthorized charger or other charging method. Terminate charging when the charging process doesn't end within specified time.
- Storage
 - Do not store the battery with metalware, water, seawater, strong acid or strong oxidizer.
 - Make the charge amount less than or equal to 50% then store at -20~40 degree C in a dry (humidity: 45~85%) place.
Since deterioration will be faster in the high temperature range than in the low temperature range, so do not keep it in the high temperature range beyond the period that is specified by the seller or owner.
 - Use insulative and adequately strong packaging material to prevent short circuit between positive and negative terminal when the packaging breaks during normal handling. Do not use conductive or easy to break packaging material..

8. EXPOSURE CONTROLS / PERSONAL PROTECTION (WHEN THE ELECTROLYTE LEAKS)

- Control parameters
ACGIH has not been mentioned control parameter of electrolyte.
- Personal protective equipment
Respiratory protection: Respirator with air cylinder, dust mask
Hand protection: Protective gloves
Eye protection: Goggle or protective glasses designed to protect against liquid splashes
Skin and body protection: Working clothes with long sleeve and long trousers

No.13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C.

TEL: 886-2-2601-0385 FAX: 886-2-2601-0460 <http://www.metco.com.tw>

9. PHYSICAL AND CHEMICAL PROPERTIES

- Appearance
Physical state : Solid
Form : Cylindrical or Prismatic or Pouch (laminated)
Color : Metallic color or black (without tube if it has tube)
Odor : No odor
Density : N/A
Boiling Point : N/A
Melting Point : N/A
Evaporation Rate : N/A
Vapor Pressure : N/A
Molecular Weight : N/A
Solubility : N/A
pH : N/A
Viscosity : N/A
Other Information : N/A

10. STABILITY AND REACTIVITY

- Stability: Normally stable unless a strong shock is applied or heated strongly
- Possibility of hazardous reactions: Damage to the container may cause leakage of contents. Contents may leak or ignite due to temperature rise.
- Conditions to avoid: Crushing or deformation, use and storage at 80 degree C or higher or at high humidity. Usage at a voltage or a current outside the rating and external short circuit.
- Incompatible materials: Conductive material such as water or metal pieces. Oxidizing agent such as bleach.
- Hazardous decomposition products: Acid or harmful gas is emitted during leakage or fire.

11. TOXICOLOGICAL INFORMATION

- **Organic Electrolyte**
- Acute toxicity:
LD₅₀, oral - Rat 2,000mg/kg or more
- Irritating nature: Irritative to skin and eye

12. ECOLOGICAL INFORMATION

- Persistence/degradability:
Since a battery cell and the internal materials remain in the environment, do not bury or throw out into the environment.

13. DISPOSAL CONSIDERATIONS

- Recommended methods for safe and environmentally preferred disposal:
Product (waste from residues)
Specified collection or disposal of lithium ion battery is required by the law like as "battery control law" in several nations. Collection or recycle of the battery is mainly imposed on battery's manufacturer or importer in the nations recycle is required.
Contaminated packaging
Neither a container nor packing is contaminated during normal use. When internal materials leaked from a battery cell contaminates, dispose as industrial wastes subject to special control.

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Mobile Energy Technology Co., Ltd

No.13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C.

TEL: 886-2-2601-0385 FAX: 886-2-2601-0460 <http://www.metco.com.tw>

14. TRANSPORT INFORMATION

In the case of transportation, avoid exposure to high temperature and prevent the formation of any condensation. Take in a cargo of them without falling, dropping and breakage. Prevent collapse of cargo piles and wet by rain. The container must be handled carefully. Do not give shocks that result in a mark of hitting on a cell. Please refer to Section 7-HANDLING AND STORAGE also.

The table mentioned below is applied to only the lithium ion rechargeable battery cell described in Section 1-PRODUCT AND COMPANY IDENTIFICATION.

	LAND TANSPORT (ADR)	SEA TRANSPORT (IMDG / IMO)	AIR TRANSPORT (IATA / ICAO)
UN Number ^a	3480	3480	3480
Proper Shipping Name ^a	LITHIUM ION BATTERIES (including lithium ion polymer batteries)	LITHIUM ION BATTERIES (including lithium ion polymer batteries)	LITHIUM ION BATTERIES (including lithium ion polymer batteries)
Hazard Class	9	9	9
Packing Group ^b	II	II	II

^a UN Number is 3481 in case of the battery is contained in equipment or packed with equipment, and Proper Shipping Name is "lithium ion batteries contained in equipment" or "lithium ion batteries Packed with equipment".

UN Number is 3171 in case of the battery is contained in vehicle which is only powered by the battery, and Proper Shipping Name is "Battery-powered vehicle".

^b Lithium ion rechargeable battery cell is not assigned to packing groups, and the packaging performance level is set out in the applicable packing instruction. Packing group II is often set out.

15. REGULATORY INFORMATION

- Regulations specifically applicable to the product:
 - Wastes Disposal and Public Cleaning Law [Japan]
 - Law for Promotion of Effective Utilization of resources [Japan]
 - US Department of Transportation 49 Code of Federal Regulations [USA]
- * About overlapping regulations, please refer to Section 15-TRANSPORT INFORMATION.

16. OTHER INFORMATION

- This safety data sheet is offered an agency who handles this product to handle it safely.
- The agency should utilize this safety data sheet effectively (put it up, educate person in charge) and take proper measures.
- The information contained in this Safety data sheet is based on the present state of knowledge and current legislation.
- This safety data sheet provides guidance on health, safety and environmental aspects of the product and should not be construed as any guarantee of technical performance or suitability for particular applications.

No.13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C.
TEL: 886-2-2601-0385 FAX: 886-2-2601-0460 <http://www.metco.com.tw>

Reference

Dangerous Goods Regulations – 63rd Edition Effective 1 January 2022: International Air Transport Association (IATA)

IMDG Code – 2020 Edition: International Maritime Organization (IMO)

The European Agreement concerning the International Carriage of Dangerous Goods by Road – 2021: The United Nations Economic Commission for Europe (UNECE)

First edition: Apr. 28, 2010

Prepared and approved by: Department of Development strategy
Corporate of Development strategy Cell Development Division

Sanyo Electric Co., Ltd. Panasonic group

Safety Data Sheet for product - Lithium ion rechargeable battery cell Reference number: SDS-IBT-00030

Establishment / Revision: Dec. 13, 2021

(*This document has been prepared taking into account regulations as of January 1, 2022)

Rechargeable Lithium Ion Battery Pack assembled by: Mobile Energy Technology Co., Ltd.

No.13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C.

TEL: 886-2-2601-0385 FAX: 886-2-2601-0460 <http://www.metco.com.tw>

Note: The reference data provide from supplier -

Corporate of Development strategy Cell Development Division

Sanyo Electric Co., Ltd. Panasonic group

Prepared and approved by Mobile Energy Technology Co., Ltd.

LITHIUM ION BATTERY SAFETY TESTING REPORT

Applicant:	Mobile Energy Technology Co., Ltd. No. 13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C. 24452 Tel: +886-2-2601-0385 ; E-mail: engineering@metco.com.tw Website: www.metco.com.tw/
Manufacturer:	Mobile Energy Technology Co., Ltd. No. 13, Kong 9th Road, 2nd Industrial Park, Linkou District, New Taipei City, Taiwan, R.O.C. 24452
Product:	Rechargeable Li-ion Battery Pack
Model:	HT-18A
Rating:	25.2 Vdc, 10.05 Ah, 253.26 Wh
Test method & Criterion	UNITED NATIONS "Recommendations on the TRANSPORT OF DANGEROUS GOODS" Manual of Tests and Criteria ST/SG/AC.10/11/Rev.7
Appearance	Plastic Shell
Verification Issuing Office Name	AnTek Certification Inc. 7F, No.351, Yangguang St. Taipei, 11491 Taiwan Tel: +886-2-2452-2256; Website: www.atclab.com.tw; E-mail: atc@atclab.com.tw
Date Received	May 11, 2022
Test Performed Date:	May 17, 2022– Jun. 21, 2022
Conclusion:	The sample has passed the test items of UN 38.3
Date of Issued:	Jul. 11, 2022
Comment:	Internal cell source: Panasonic / NCR18650GA

Prepared by:

Ann Kuo
Project handler

Approved by:

Nick Wu
Technical reviewer

SAFETY DATA SHEETS

Section 1. Identification of the substance/mixture and of the company/undertaking

1.1 Product identifier

Product Identification: Rechargeable Lithium Battery Module
Rechargeable Lithium Battery Module Cell Configuration: 7S3P
Customer Model Name: Rechargeable Lithium Battery Module
Customer P/N: 77172679
Product Manufacturing Place: Taiwan

1.2 Relevant identified uses of the substance or mixture and uses advised against

Relevant identified uses of product: Application of LEV

1.3 Details of the supplier of the safety data sheet

Manufacturer: STL Technology Co., Ltd.
Address: No. 1, West 15th Street, Cianjhen Dist. Kaohsiung, 806011 Taiwan, R.O.C.
Telephone number: +886-7-8411501
FAX: +886-7-8111314

1.4 Emergency phone number:

Call CHEMTREC
Domestic North America 1- 800-424-9300
International, Call 1-703-741-5970

Section 2. Hazard(s) Identification

2.1 GHS Classification Categories: Not applicable with normal use.

2.2 Classification of the substance or mixture.

2.2.1 Preparation Hazards and Classification:

The product is a Lithium ion battery and is therefore classified as an article and is not hazardous when used according to the recommendations of the manufacturer. The hazard is associated with the contents of the battery. Under recommended use conditions, the electrode materials and liquid electrolyte are non-reactive provided that the battery integrity remains and the seals remain intact. The potential for exposure should not exist unless the cell in the battery leaks, is exposed to high temperatures or is mechanically, electrically or physically abused/damaged. If the cell in the battery is compromised and starts to leak, based upon the battery ingredients, the contents are classified as Hazardous.

2.2.2 Hazard Summary:

Physical hazards: Not classified for physical hazards.
Health hazards: Not classified for health hazards.
Environmental hazards: Not classified for hazards to the environment.
Specific hazards: Exposure to contents of an open or damaged cell or battery: contact with this material will cause burns to the skin, eyes and mucous membranes.

SAFETY DATA SHEETS

May cause sensitization by skin contact.

2.2 Label elements

2.3 Other Hazards

2.3.1 Appearance, Color and Odor: Solid object with no odor.

2.3.2 Primary Routes(s) of Exposure: thermally, electrically or physically abused to the point of compromising the enclosure. If this occurs, exposure to the electrolyte solution contained within can occur by inhalation, ingestion, eye contact and skin contact.

2.3.3 Potential Health Effect(s):

2.3.3.1 Acute (short term): see Section 8 for exposure controls.

In the event that this cell or pack has been ruptured, the electrolyte solution contained within the cell would be corrosive and can cause burns to skin and eyes.

Inhalation: Inhalation of materials from a sealed cell is not an expected route of exposure. Vapors or mists from a ruptured cell may cause respiratory irritation.

Ingestion: Swallowing of materials from a sealed cell is not an expected route of exposure.

Swallowing the contents of an open cell can cause serious chemical burns to mouth, esophagus, and gastrointestinal tract.

Skin: Contact between the cell and skin will not cause any harm. Skin contact with the contents of an open cell can cause severe irritation or burns to the skin.

Eye: Contact between the cell and the eye will not cause any harm. Eye contact with the contents of an open cell can cause severe irritation or burns to the eye.

2.3.3.2 CHRONIC (long term): see Section 11 for additional toxicological data.

2.3.4 Medical Conditions Aggravated by Exposure: Not Available.

2.3.5 Interactions with other chemicals: Immersion in high conductivity liquids may cause corrosion and breaching of the cell or battery enclosure. The electrolyte solution inside of the cells may react with alkaline (basic) materials and present a flammability hazard.

2.3.6 Potential Environmental Effects: Not Available.

SAFETY DATA SHEETS

Section 3. Composition/information on ingredients

- 3.1 Substance:** Lithium Ion Battery
- 3.2 CAS number:** Not specified
- 3.3 Cases:** ABS · not dangerous
- 3.4 Printed Circuit Board Assembly :** Not dangerous
- 3.5 Lithium Ion Cell:** Panasonic NCR18650GA 3300mAh

Information about the chemical nature of product: *1

Portion	Material name	Concentration range (wt %)
Positive electrode	Lithium transition metal oxidate (Li[M] _m [O] _n *2)	20~60
Positive electrode's base	Aluminum	1~10
Negative electrode	Carbon	10~30
Negative electrode's base	Copper	1~15
Electrolyte	Organic electrolyte principally involves ester carbonate	5~25
Outer case	Aluminum, iron, aluminum laminated plastic	1~30

*1 Not every product includes all of these materials.

*2 The letter M means transition metal and candidates of M are Co, Mn, Ni and Al. One compound includes one or more of these metals and one product includes one or more of the compounds. The letter m and n means the number of atoms.

Section 4. First-aid measures

4.1 Description of first aid measures

The hazardous components of this cell or battery are contained within a sealed unit. The following measures are only applicable if exposure has occurred to components when a cell or battery leaks, is exposed to high temperatures or is mechanically, electrically or physically abused/damaged. The hazardous contents are caustic alkaline electrolytes contained in cells with lithium metal oxide cathodes, graphite and carbon anodes and Polyvinylidenfluoride binders.

Inhalation: Not anticipated. If battery is leaking, contents may be irritating to respiratory passages.

Remove to fresh air. Contact physician if irritation persists.

Skin: Not anticipated. If battery is leaking, irrigate exposed skin with copious amounts of clear, tepid water for at least 15 minutes. If irritation, injury or pain persists, consult a physician.

Eye Contact: Not anticipated. If battery is leaking and material contacts eyes, flush with copious amounts of clear, tepid water for at least 30 minutes. Contact physician at once.

Ingestion: Not anticipated. Consult a physician immediately for treatment.

PROTECTION FOR FIRST

AIDERS: Do not enter corrosive vapor contaminated areas without a respirator or Self Contained Breathing Apparatus. Wear adequate personal protective equipment as indicated in Section 8.

FIRST AID FACILITIES: Eye wash bottle, fountain, safety showers or at least a source of running water are required in the area where the product is used.

SAFETY DATA SHEETS

4.2 MOST IMPORTANT SYMPTOMS & EFFECTS, ACUTE & DELAYED, CAUSED BY EXPOSURE:

ACUTE: The contents of the battery are rated as corrosive. Ingestion of the electrolyte could lead to severe gastrointestinal tract irritation with nausea, vomiting and potentially burns. Inhalation of vapors may lead to severe irritation of the mouth and upper respiratory tract with a burning sensation, pain, burns and inflammation in the nose and throat; there may also be coughing or difficulty breathing. Eye contact may lead to severe eye irritation or in worst case scenario irreversible damage and possible eye burns. Skin contact may lead to irritation and possible skin burns.

CHRONIC: Skin contact may aggravate/exacerbate existing skin conditions, such as dermatitis. Chronic inhalation may lead to the same symptoms as listed for acute inhalation above.

4.3 Indication of any immediate medical attention and special treatment needed

ADVICE TO DOCTOR: Treat symptomatically if the person comes into contact with the corrosive electrolyte liquid contents of a damaged battery.

Section 5. Fire-fighting measures

5.1 Extinguishing media

5.1.1 Suitable extinguishing media: Dry chemical, carbon dioxide and foam. Water acts as a cooling agent.

5.1.2 Unsuitable extinguishing media: Strong oxidizing agents, strong reducing agents, strong acids and strong alkalis. Despite water incompatibility, water is the most effective firefighting tool to control the spread of fire to other cells and batteries and combustibles.

5.1.3 Explosion Data: Closed containers may explode, burst, rupture or vent when exposed to temperatures above 120°C (248°F).

5.2 Special hazards arising from the substance or mixture:

The interaction of water vapor and exposed lithium hexafluorophosphate (LiPF₆) may result in the generation of hydrogen and hydrogen fluoride (HF) gas. Contact with battery electrolyte may be irritating to skin, eyes and mucous membranes. Thermal degradation may produce hazardous fumes of lithium, cobalt and manganese, hydrofluoric acid, hydrogen and oxides of carbon, aluminum, lithium, copper and cobalt as well and smoke and irritating, corrosive and/or toxic gases. Fumes may cause dizziness or suffocation.

5.3 Advice for firefighters:

In case of fire where lithium-ion cells and batteries are present, flood the area with water. If any cells or batteries are burning, water may not extinguish them, but will cool the adjacent cells or batteries and control the spread of fire. Carbon Dioxide, dry chemical and foam extinguishers may be preferred for small fires, but also may not extinguish burning lithium-ion cells or batteries. Burning cells or batteries will burn themselves out. Virtually all fires involving lithium-ion cells and batteries can be controlled with water. When water is used, however, hydrogen gas may be evolved which can form an explosive

SAFETY DATA SHEETS

mixture with air. LITH-X (powdered graphite) or copper powder fire extinguishers, sand, dry ground dolomite or soda ash may also be used. These materials act as smothering agents.

5.4 Protective Equipment and Precautions for firefighters:

In case of a fire and the release of hydrogen fluoride, it is critical to protect the skin from any contact. Fire fighters should wear a self-contained breathing apparatus. Burning lithium-ion cells and batteries can produce toxic fumes including hydrogen fluoride (HF), oxides of carbon, aluminum, lithium, copper and cobalt. Volatile phosphorous penta fluoride may form at temperatures above 110°C (230°F). Wear adequate personal protective equipment as indicated in Section 8.

Section 6. Accidental release measures

6.1. Personal precautions, protective equipment and emergency procedures:

6.1.1. As an immediate precautionary measure, isolate spill or leak area for at least 25 meters (75 feet) in all directions. Keep unauthorized personnel away. Stay upwind. Keep out of low areas. Ventilate closed areas before entering. Wear adequate personal protective equipment as indicated in Section 8.

6.2. Environmental precautions:

6.2.1. Absorb spilled material with non-reactive absorbent such as vermiculite, clay or earth. Prevent from migration into soil, sewers and natural waterways – inform local authorities if this occurs.

6.3. Methods and material for containment and cleaning up:

6.3.1 Evacuate spill area immediately and remove sources of ignition. Do NOT touch spilled material. Cleanup personnel must be trained in the safe handling of this product. Spills may be absorbed on non-reactive absorbents such as vermiculite. Place cells or batteries into individual plastic bags and then place into appropriate containers and close tightly for disposal. Ensure that cleanup procedures do not expose spilled material to any moisture. Immediately transport closed containers outside. Lined steel drums are suitable for storage of damaged cells or batteries until proper disposal can be arranged.

Section 7. Handling and storage

7.1. Precautions for safe handling

- Do not crush, pierce, short (+) and (-) battery terminals with conductive (i.e. metal) goods.
- Do not directly heat or solder.
- Do not expose to extreme heat or fire
- Do not mix batteries of different types and brands.
- Do not mix new and used batteries.
- Do not immerse cell or battery in liquids as there is a danger of short circuiting the cells or batteries.
- Do not incinerate the cells or batteries as there is a danger of explosion.

SAFETY DATA SHEETS

- Do not use or charge damaged, defective or deformed cells or batteries.
- Keep batteries in non conductive (i.e. plastic) trays.
- This package must be handled with care and a flammability hazard exists, if the package is damaged.
- Do not damage or mishandle this package. If the package is damaged. The package must be inspected and if necessary, batteries must be repacked so as to prevent short circuit.
- Observe good industrial hygiene practices. Wash hands thoroughly after handling.

7.2. Conditions for safe storage, including any incompatibilities

7.2.1. Conditions for safe storage: Store in a cool, dry, well-ventilated area, out of direct sunlight and away from heat and ignition sources. To minimize any adverse effects on cell and/or battery performance, it is recommended that the cells and/or batteries be kept at room temperature (25°C +/- 5°C). Elevated temperatures can result in shortened cell and/or battery life. Keep out of reach of children. Store away from incompatible materials, see Section 10 of the SDS.

7.2.2. Incompatibilities: Water, strong oxidizing agents, strong reducing agents, strong acids and strong alkalis.

7.3 Specific end use(s): The Lithium-ion battery pack for this products such as NB 、 Power tool 、 E-bike 、 E-scooter 、 E-Car 、 UPS 、 ESS...

Section 8. Exposure controls / personal protection

8.1. Exposure Control Measures

8.1.1. Exposure Limit Values: Airborne exposures to hazardous substances are not expected when the cells or batteries are used for their intended purposes. Exposure standards are not applicable to the sealed articles.

8.1.2. Biological Monitoring: Not applicable.

8.1.3. Control Banding: Not applicable.

8.1.4. Recommended monitoring procedures: Follow standard monitoring procedures.

8.1.5. Derived no-effect level (DNEL): Not applicable.

8.1.6. Derived minimal effect level (DMEL): Not applicable.

8.1.7. Predicted no-effect concentrations (PNECs): Not applicable

8.2. Engineering Controls:

8.2.1. Engineering Controls: Special ventilation is not required when using these products in normal use scenarios. Ventilation is required if there is leakage from the cell or battery.

8.2.2. Individual Protection Measures

8.2.2.1. Eye and Face protection: Eye protection is not required when handling cells or batteries during normal use. Wear safety glasses/goggles if handling a leaking or ruptured cell or battery.

8.2.2.2. Skin (Hand) protection: Hand protection is not required when handling the cell or battery

SAFETY DATA SHEETS

during normal use. PVC gloves are recommended when dealing with a leaking or ruptured cell or battery.

8.2.2.3. Skin (clothing) protection: Skin protection is not required when handling the cell or battery during normal use. Wear long sleeved clothing to avoid skin contact if handling a leaking or ruptured cell or battery. Soiled clothing should be washed with detergent prior to re-use.

8.2.2.4. Respiratory protection: During routine operation, a respirator is not required. However, if dealing with an electrolyte leakage and irritating vapors are generated, an approved half face inorganic vapor and gas/acid/particulate respirator is required.

8.2.2.5. Thermal Protection: Not applicable.

8.2.2.6. Other Protective Equipment: Have a safety shower or eye wash station readily available.

8.2.3. Hygiene Measures: Do not eat, drink or smoke in work areas. Avoid storing food, drink or tobacco near the product. Practice and maintain good housekeeping.

8.2.4. Environmental exposure controls: Avoid release to the environment.

Respiratory Protection	Hand Protection	Eye Protection	Other
			
In all fire situations, use self-contained breathing apparatus.	In the event of leaking or ruptured cells or batteries, wear gloves.	Safety glasses are recommended in case of leaking or ruptured cells or batteries.	In the event of leaking or ruptured cells or batteries, wear protective clothing.

SAFETY DATA SHEETS

Section 9. Physical and chemical properties

State	Solid, Sealed Unit	Upper/lower flammability or explosive limits	Not Applicable
Physical state	Solid object with no odor	Vapor pressure	Not Applicable
Color	Black		
Odor Type	Odorless	Vapor density	Not Applicable
Odor threshold	Not Applicable	Relative density	Not Applicable
pH	Not Applicable	Solubility in Water	Insoluble
Vapor pressure	Not Applicable	Partition coefficient: n-octanol/water	Not Applicable
Melting point/freezing point	Not Applicable	Auto-ignition temperature	Not Applicable
Flash point	Not Applicable	Decomposition temperature	90°C
Evaporation rate	Not Applicable	Kinematic viscosity	Not Applicable
Flammability (solid, gas)	Not Applicable	Particle characteristics	Not Applicable

Section 10. Stability and reactivity

10.1 Chemical Stability: The cells or batteries are stable under normal conditions of use, storage and transport.

10.2 Possibility of hazardous reactions: Keep away from water, strong oxidizing agents, strong reducing agents, strong acids and strong alkalis. Reaction of the leaking electrolyte materials with water may produce flammable and explosive hydrogen gas as well as corrosive hydrogen fluoride gas. Hazardous polymerization does not occur.

10.3 Incompatible materials: Do not immerse in water or other high conductivity liquids.

10.4 Hazardous decomposition products: May decompose to produce hydrogen fluoride, phosphorus oxides, sulfur oxides, sulfuric acid, lithium hydroxide, carbon monoxide and carbon dioxide.

10.5 Reactivity: The cells or batteries do not pose any further reactivity hazards other than those listed in the following sub-sections.

10.6 Conditions to avoid: Avoid exposing the cells or batteries to fire or temperatures above 80°C. Do not disassemble, crush, short circuit, puncture, immerse in liquid, burn, expose to flame or install with incorrect polarity. Avoid mechanical, physical or electrical abuse.

Section 11. Toxicological Information

SAFETY DATA SHEETS

11.1. Information on toxicological effects:

The hazardous components of the cell or battery are contained within a sealed unit. Under recommended use conditions, the electrode materials and liquid electrolyte are non-reactive provided that the cell or battery integrity remains and the seals remain intact. The potential for exposure should not exist unless the battery leaks, is exposed to high temperature or is mechanically, electrically or physically abused/damaged. **The following toxicology data is in respect to if a person comes into contact with the electrolyte.**

11.2. Acute Toxicity:

11.2.1. Swallowed: The electrolyte contained within the cell or battery is a corrosive liquid. Ingestion of this electrolyte would be harmful. Swallowing may result in nausea, vomiting, diarrhea, abdominal pain and chemical burns to the gastrointestinal tract. During normal usage ingestion should not be a means of exposure.

11.2.2. Eye: The electrolyte contained within the cell or battery is a corrosive liquid and it is expected that it would cause irreversible damage to the eyes. Contact may cause corneal burns. Effects may be slow to heal after eye contact. Correct handling procedures incorporating appropriate eye protection should minimize the risk of eye irritation.

11.2.3. Skin: The electrolyte contained within the cell or battery is a corrosive liquid and it is expected that it would cause skin burns or severe irritation to the skin if not washed off immediately. Correct handling procedures should minimize the risk of skin irritation. People with pre-existing skin conditions, such as dermatitis, should take extreme care so as not to exacerbate the condition.

11.2.4. Inhaled: Inhalation of vapors from a leaking cell or battery is expected to cause severe irritation of the mouth and upper respiratory tract with a burning sensation, pain, burns and inflammation in the nose and throat; there may also be coughing or difficulty breathing.

11.3. Skin Corrosion/Irritation: The electrolyte contained within the cell or battery is classified as a corrosive liquid and is expected to exhibit Dermal Corrosivity /Irritation.

11.4. Serious Eye Damage/Irritation: The electrolyte contained within the cell or battery is classified as a corrosive liquid and is expected to exhibit serious Damage/Corrosivity.

11.5. Respiratory or Skin Sensitization: The electrolyte contained within the cell or battery is not expected to be a skin sensitizer according to OECD test 406, based on the available data and the known hazards of the components. The electrolyte contained within the battery is not expected to be a respiratory tract sensitizer, based on the available data and the known hazards of the components.

11.6. Germ Cell Mutagenicity: The electrolyte contained within the cell or battery is not expected to be mutagenic according to test such as OECD tests 471, 475, 476, 478 and 479, based on the available data and the known hazards of the components.

11.7. Carcinogenicity: The electrolyte contained within the cell or battery is not expected to be a carcinogen. The cathode contains Cobalt and Nickel components. These components are classified as

SAFETY DATA SHEETS

IARC 2B – possibly carcinogenic to humans, however they do not pose a threat when contained in the cell or battery sealed unit.

- 11.8. Reproductive Toxicity:** The electrolyte contained within the cell or battery is not expected to be a reproductive hazard according to test such as OECD tests 414 and 421, based on the available data and the known hazards of the components.
- 11.9. Specific Target Organ Toxicity (STOT) – Single Exposure:** The electrolyte contained within the cell or battery is corrosive and is expect to cause respiratory irritation by inhalation. Inhalation of vapors may lead to severe irritation of the mouth and upper respiratory tract with a burning sensation, pain, burns and inflammation in the nose and throat; there may also be coughing or difficulty breathing.
- 11.10. Specific Target Organ Toxicity (STOT) – Repeated Exposure:** The cells or batteries are not expected to cause organ damage from prolonged or repeated exposure according to tests such as OECD tests 410 and 412, based on the available data and the known hazards of the components.
- 11.11. Aspiration Hazard:** The cells or batteries are not classified as an aspiration hazard, based on the available data and the known hazards of the components. However, due to the corrosive nature of the product if swallowed, do NOT induce vomiting. If vomiting has occurred after ingestion the person should be observed to ensure that aspiration into the lungs has not occurred and assessed for chemical burns to the gastrointestinal and respiratory tracts.
- 11.12 Information on other hazards:**
- 11.12.1 Endocrine disrupting properties:** Not available.
- 11.12.2 Other information:** No additional data is available

Section 12. Ecological Information

- 12.1. Ecotoxicity:** The sealed cell or battery does not pose an Ecotoxicity hazard. Cells or batteries under normal use conditions pose no ecotoxicity hazard. In the case of a broken or damaged cell or battery and leakage of the electrolyte, it will react with water and potentially cause damage to flora and fauna if not disposed of properly. See Section 13 of this SDS for proper disposal considerations.
- 12.2. Persistence and degradability:** There is currently no data available.
- 12.3. Bio accumulative potential:** There is currently no data available.
- 12.3.1. Partition coefficient n-octanol/water (log Kow):** Not applicable.
- 12.3.2. Bio concentration factor (BCF):** Not available.
- 12.4. Mobility in soil:** There is currently no data available.
- 12.5. Results of PBT and vPvB assessment:** Not a PBT or vPvB substance or mixture.
- 12.6 Endocrine disrupting properties:** Not available.
- 12.7. Other adverse effects:** Solid cells and batteries released into the natural environment will slowly degrade and may release harmful or toxic substances. Cells and batteries are not intended to be released into water or on land but should be disposed or recycled according to local regulations. See section 13 of this SDS for Disposal Considerations.

SAFETY DATA SHEETS

Section 13. Disposal Consideration:

- 13.1. Waste treatment methods:** Cell and battery recycling is encouraged. Cells and batteries should not be released into the environment, do **NOT** dump into any sewers, on the ground or into any body of water. Do not dispose of in fire. Used cells and batteries should be stored in their original packaging, a plastic bag or with their terminals/contacts taped, to minimize the potential for short-circuiting to occur. Cells and batteries should be fully discharged before being sent for recycling. Do not store used cells or batteries near heat sources, chemicals or food. Do not store or transport used lithium-ion cells or batteries with lead acid batteries as they have different regulatory requirements. Do not break open or damage lithium-ion cells or batteries prior to disposal. Care should be taken at all times to ensure that used cells or batteries are not damaged during storage or transport. Store material for disposal as indicated in Section 7 Handling and Storage
- 13.2. Classification of the waste to comply with Transport Regulations:** Spent lithium-ion cells and batteries are not considered hazardous waste. Lithium-ion cells and batteries involved in a fire may be considered to be hazardous waste and should be classified as such. Damaged lithium-ion cells and batteries are explicitly prohibited from transport by air.
- 13.3. Classification of Packaging materials:** Unsoiled excess packaging should be disposed of according to any applicable recycling regulations and is not considered hazardous waste. Soiled packaging or packaging exposed to the interior of a lithium-ion cell or battery pack should be considered hazardous waste and disposed of according to local hazardous waste rules and regulations.

Section 14. Transport Information

Nominal Voltage(DCV)	Nominal Capacity(mAh)	Power(Wh)
25.2	10050	253.2

14.1 UN number: UN3480

14.2 UN proper shipping name: Lithium ion batteries

14.3 Transport hazard class: 9

14.4 Packing group: II

14.5 Environmental hazards:

Marine Pollutant: No.

14.6 Special precautions for user

14.6.1 Lithium ion batteries only transport by air in accordance with PI965 at a state of charge (SOC) not to exceed 30 percent of rated design capacity.

14.6.2 IATA/IMDG ERG Code: 12FZ

SAFETY DATA SHEETS

14.6.3 DOT ERG Code: 147

14.7 Maritime transport in bulk according to IMO instruments: Not Applicable.

Other information:

- **IATA(Air):**
 Packing Instruction: 965
 Section: IA
 (Special Provision A88、A99、A154、A164、A183、A201、A213、A331、A334 and A802 may apply depends on content goods of the transport).
- **IMDG(Sea)**
 Packing Instruction: P903
 (Packing Instruction P908、P909、P910、P911、LP903、LP904、LP905 & LP906 and Special Provision 188、230、310、348、376、377、384 & 387 may apply depends on content goods of the transport).
- US Department of Transportation (DOT) 49 code of Federal Regulations [USA] International Civil Aviation Administration (ICAO)
- There is no hazards in accordance with the UN recommendations tests (Manual of Tests and Criteria, Part III, sub-section 38.3)

No	ITEMS	RESULT	REMARKS
1	Altitude Simulation	Pass	
2	Thermal Shock	Pass	
3	Vibration	Pass	
4	Shock	Pass	
5	External Short	Pass	
6	Impact/Crush	Pass	For cell only
7	Overcharge	Pass	
8	Forced Discharge	Pass	For cell only

Section 15. Regulatory Information

15.1. Safety, health and environmental regulations/legislation specific for the substance or mixture:

15.1.1. International Information

- 15.1.1.1. Montreal Protocol: Not applicable
- 15.1.1.2. Stockholm Convention: Not applicable
- 15.1.1.3. Rotterdam Convention: Not applicable
- 15.1.1.4. Basel Convention: Not applicable
- 15.1.1.5. Marpol Convention: Not applicable

Page 12 of 14

SAFETY DATA SHEETS

15.1.1.6. UN Transportation of Dangerous Goods: All cells and batteries have passed the applicable testing.

15.1.2. United States Federal and State Regulations:

TSCA Status: All ingredients in these products are listed on the TSCA inventory.

OSHA: These products do not meet criteria as per Part 1910.1200, manufactured article.

SARA EPA Title III: None.

Sec. 302/304: None.

Sec. 311/312: None.

Sec. 313: None.

CERCLA RQ: None.

15.1.3. EC Classification for the Substance/Preparation:

These products are not classified as hazardous according to Regulation (EC) No. 1272/2008.

Keep out of the reach of children.

15.1.3.1. EU Regulations:

- Regulation (EC) No. 1005/2009 on substances that deplete the ozone layer, Annex I: Not listed.
- Regulation (EC) No. 1005/2009 on substances that deplete the ozone layer, Annex II: Not listed.
- Regulation (EC) No. 850/2004 on persistent organic pollutants, Annex I as amended: Not listed.
- Regulation (EC) No. 689/2008 concerning the export and import of dangerous chemicals, Annex I, part 1 as amended: Not listed.
- Regulation (EC) No. 689/2008 concerning the export and import of dangerous chemicals, Annex I, part 2 as amended: Not listed.
- Regulation (EC) No. 689/2008 concerning the export and import of dangerous chemicals, Annex I, part 3 as amended: Not listed.
- Regulation (EC) No. 689/2008 concerning the export and import of dangerous chemicals, Annex V as amended: Not listed.
- Regulation (EC) No. 166/2006, REACH Article 59(10) Candidate List as currently published by ECHA: Not listed.

15.1.3.2. EU Authorizations:

Regulation (EC) No. 1907/2006, REACH Annex XIV Substances subject to authorization, as amended: Not listed

15.1.3.3. EU Restrictions on use:

- Regulation (EC) No. 1907/2006, REACH Annex XVII
- Directive 2004/37/EC: on the safety and health of pregnant workers and workers who have recently given birth or are breastfeeding: Not listed

15.1.3.4. Other EU Regulations:

- Directive 96/82/EC (Seveso II) on the control of major accident hazards involving dangerous substances: Not listed.

SAFETY DATA SHEETS

- Directive 94/33/EC on the protection of young people at work: Not listed.

This Safety Data Sheet complies with the requirements of Regulation (EC) No. 1907/2006.

15.1.4. Chinese Regulations:

- General Rule for Classification and Hazard Communication of Chemicals (GB 13690-2009):

Specifies the classification, labeling and hazard communication of chemicals in compliance with the GHS standard for chemical production sites and labeling of consumer goods.

- General Rule for Preparation of Precautionary Labels for Chemicals (GB 15258-2009): Specifies the relevant application methods of precautionary labels for chemicals.

Safety Data Sheet for Chemical Products Content and Order of Sections (GB/T 16483-2008)

15.2. Chemical Safety Assessment: Not applicable.

Section 16. Other Information:

The information contained in this Safety data sheet is based on the present state of knowledge and current legislation. This safety data sheet provides guidance on health, safety and environmental aspects of the product and should not be construed as any guarantee of technical performance or suitability for particular applications.

End of Safety Data Sheet

Lithium Ion Battery UN38.3 Test Report

Applicant:	STL Technology Co., Ltd. No. 1, West 15th St., Cianjhen Dist., Kaohsiung City 806011, Taiwan Tel: +886-7-841-1501 ; E-mail: stl@stl-tech.com Website: http://www.stl-tech.com/
Manufacturer:	STL Technology Co., Ltd. No. 1, West 15th St., Cianjhen Dist., Kaohsiung City 806011, Taiwan
Product:	Rechargeable Li-ion Battery Pack
Model:	77172679
Rating:	25.2V / 10.05Ah / 253.2Wh
Test method & Criterion	UNITED NATIONS "Recommendations on the TRANSPORT OF DANGEROUS GOODS" Manual of Tests and Criteria ST/SG/AC.10/11/Rev.7
Appearance	Black plastics shell
Verification Issuing office Name	STL Laboratory No. 1, West 15th St., Cianjhen Dist., Kaohsiung City 806011, Taiwan Tel: +886-7-841-1501 ; E-mail: stl@stl-tech.com
Date Received	20230116
Test performed Date:	20230122 ~ 20230309
Conclusion:	The sample has passed the test items of UN 38.3
Data of issued	20230323
Comment:	Cell type : Panasonic NCR18650GA

Prepared by:

Checked by

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Prepared by:

Approved by

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The following symbols are used in the scooter to identify warnings, mandatory actions and prohibited actions. It is very important that you read and understand them completely.

Read and follow the information in the instruction manual.

Caution. Observing notes and accompanying documents

Scooter Class A

It is classified in category A according to EN 12184. It is compact, maneuverable and not necessarily able to overcome obstacles outdoors

For ambient conditions

Protection class II

Charger IPX0 "Protect from moisture" / Scooter IPX4 "Splashproof"

Does not meet the ISO 7176-19 standard and cannot be used as a seat in a motor vehicle.
Labeling on the product.

Devices for decommissioning the drive system
Warning: Switch on the drive system again before leaving a user unattended.

Charge the batteries fully before use.
Remove the key from an unattended scooter.

Year of manufacture see label on the product.

Heartway Medical Products Co., Ltd.

No. 6, Gongyequ 25th Rd., Baoshan Vil., Nantun Dist.,
Taichung City 408017, Taiwan (R.O.C.)
TEL: +886-4-2358-3232 ext: 306

Europäischer Bevollmächtigter / Authorized Representative
SUNGO Europe B.V. Address: Fascinatio Boulevard 522,
Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands

Skyline

your guide in travel mobility