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Material Safety Data Sheet (MSDS) for passengerships and airlines.

Version: V1.4

MSDS

MATERIAL SAFETY DATA SHEET

Prepared For : Changzhou Fengyu New Energy Technology Co., Ltd.
No. 35 Leshan Road, Xinbei District, Changzhou City,
Jiangsu Province, China

Prepared By : Shenzhen LCS Compliance Testing Laboratory Ltd.
101, 201 Building A and 301 Building C, Juji Industrial
Park, Yabianxueziwei, Shajing Street, Baoan District,
Shenzhen, Guangdong, China

Issue Date : 2020.01.14

Report Number : LCS190806165ASD

Written by: Taylor zhang

Approved by: [Signature]

Material Safety Data Sheet

REPORT NO.: LCS190806165ASD

* The MSDS is prepared based on the information provided by client. The contents and formats of this MSDS are revised as per client's request.

Section 1-Chemical Product and Company Identification

Product Name	Polymer Lithium-Ion Battery
Model	FY24-12
Trade Mark	N/A
Ratings	24V, 12Ah, 288Wh
Weight	2587.8g
Manufacturer	Changzhou Fengyu New Energy Technology Co., Ltd.
Manufacturer address	No. 35 Leshan Road, Xinbei District, Changzhou City, Jiangsu Province, China
Emergency Telephone	+86-519-83859198
Fax	+86-519-83859098

Section 2- Composition Information

Chemical Composition	CAS No.	Weight (%)
Aluminum Foil(AI)	7429-90-5	5
Copper Foil(Cu)	7440-50-8	7-10
Lithium Cobalt Oxide(LiCoO ₂)	12190-79-3	2-3
Nickel(Ni)	7440-02-0	2-5
Ethylene carbonate(EC C ₃ H ₄ O ₃)	96-49-1	5
Dimethyl carbonate(DMC C ₃ H ₆ O ₃)	616-38-6	2-5
LiNixCoyMn1-x-yo2	N/A	58-62
Graphite (C)	7782-42-5	12-15
Polyvinylidene fluoride (PVDF C ₂ H ₂ F ₂) _n	24937-79-9	6

*** The exact percentage (concentration) of composition has been withheld as a trade secret.

Section 3- Hazards Identification

Emergency overview:	N/A
Classification according to GHS	Not a dangerous substance according to GHS
Label elements:	

Material Safety Data Sheet

REPORT NO.: LCS190806165ASD

Hazard pictogram(s)	Not Available
Signal word	Not Available
Hazard statement(s)	Not Available
Precautionary statement(s):	
Prevention	Not Available
Response	Not Available
Disposal	Not Available
Environmental hazards:	No relevant information
Important symptoms:	See section 11 for more information
Section 4- First Aid Measures	
Eye contact	Flush eyes with plenty of water for least 15 minutes, occasionally lifting the upper and lower eyelids. Get medical aid.
Skin contact	Remove contaminated clothes and rinse skin with plenty of water or shower for 15 minutes. Get medical aid.
Inhalation	Remove from exposure and move to fresh air immediately. Use oxygen if available.
Ingestion	Give at least 2 glasses of milk or water. Induce vomiting unless patient is unconscious. Call a physician.
Section 5- Fire Fighting Measures	
Flash Point	N/A
Auto-Ignition Temperature	N/A
Extinguishing Media	H ₂ O, CO ₂
Special Fire-Fighting Procedures	Self-contained breathing apparatus
Unusual Fire and Explosion Hazards	Cell may vent when subjected to excessive heat-exposing battery contents
Hazardous Combustion Products	Carbon monoxide, carbon dioxide, lithium oxide fumes.
Section 6- Accidental Release Measures	
Personal precautions, protective equipment and emergency procedures:	
If the battery is released, remove personnel from area until fumes dissipate. Provide maximum ventilation to clear out hazardous gases. The preferred response is to leave the area and allow the vapors to dissipate. Avoid skin and eyes contact or inhalation of vapors. Remove spilled liquid with absorbent and incinerated. If leakage of the battery happens, liquid could be absorbed with sand, earth or other inert substance and contaminated area should be ventilated meantime.	

Material Safety Data Sheet

REPORT NO.: LCS190806165ASD

Environment precautions:

Do not allow product to reach sewage system or any water source.
Inform respective authorities in case of seepage into water course or sewage system.
Do not allow to enter sewers/ surface or ground water.

Methods and material for containment and cleaning up:

If battery casing is dismantled, small amounts of electrolyte may leak. Collect all released material in a plastic lined container. Dispose off according to the local law and rules. Avoid leached substances to get into the earth, canalization or waters.

Section 7- Handling and Storage

Handling	The battery should not be opened, destroyed or incinerate, since they may leak or rupture and release to the environment the ingredients that they contain in the hermetically sealed container. Do not short circuit terminals, or over charge the battery, forced over-discharge, throw to fire. Do not crush or puncture the battery, or immerse in liquids.
Storage	Avoid mechanical or electrical abuse. Storage preferably in cool, dry and ventilated area, which is subject to little temperature change. Storage at high temperatures should be avoided. Do not place the battery near heating equipment, nor expose to direct sunlight for long periods.
Other Precautions	The battery may explode or cause burns, if disassembled, crushed or exposed to fire or high temperatures. Do not short or install with incorrect polarity.

Section 8- Exposure Controls/Personal Protection

Engineering Controls	Use local exhaust ventilation or other engineering controls to control sources of dust, mist, fumes and vapor. Keep away from heat and open flame. Store in a cool, dry place.
Personal Protective Equipment	Respiratory Protection: Not necessary under normal conditions. Skin and body Protection: Not necessary under normal conditions, Wear suitable protective clothing and gloves if handling an open or leaking battery. Hand protection: Wear suitable gloves if handling an open or leaking battery. Eye Protection: Not necessary under normal conditions, Wear safety glasses if handling an open or leaking battery.
Other Protective Equipment	Have a safety shower and eye wash fountain readily available in the immediate work area.
Hygiene Measures	Do not eat, drink, or smoke in work area. Maintain good housekeeping.

Section 9- Physical and Chemical Properties

Form	Solid
Color	Black

Material Safety Data Sheet

REPORT NO.: LCS190806165ASD

Odour	Not Available
pH	Not Available
Melting point/freezing point	Not Available
Boiling Point and Boiling range	Not Available
Flash Point	Not Available
Upper/lower flammability or explosive limits	Not Available
Vapor Pressure	Not Available
Vapor Density	Not Available
Relative density	Not Available
Solubility in Water	Not Available
Auto-ignition temperature	Not Available
Decomposition temperature	Not Available
Evaporation rate	Not Available
Flammability (soil, gas)	Not Available
Viscosity	Not Available
Section 10- Stability and reactivity	
Stability	The product is stable under conditions described Section 7
Conditions to Avoid	Heat above 70°C or incinerate. Deform, Mutilate, Crush, Disassemble, Overcharge, Short circuit, Expose over a long period to humid conditions.
Incompatible Materials	Oxidizing agents, acid, base.
Hazardous Decomposition Products	Carbon monoxide, carbon dioxide, lithium oxide fumes.
Possibility of Hazardous Reaction	Not Available
Section 11 – Toxicological Information	
Irritation	Risk of irritation occurs only if the cell is mechanically, thermally or electrically abused to the point of compromising the enclosure. If this occurs, irritation to the skin, eyes and respiratory tract may occur.
Sensitization	Not Available
Neurological Effects	Not Available

Material Safety Data Sheet

REPORT NO.: LCS190806165ASD

Teratogenicity	Not Available	
Reproductive Toxicity	Not Available	
Mutagenicity (Genetic Effects)	Not Available	
Toxicologically Synergistic Materials	Not Available	
Section 12- Ecological Information		
Ecological Toxicity	Not Available	
Mobility in soil	Not Available	
Persistence and Degradability	Not Available	
Bioaccumulation potential	Not Available	
Other Adverse Effects	Not Available	
Section 13- Disposal Considerations		
Product disposal recommendation	Observe local, state and federal laws and regulations.	
Uncleaned packaging recommendation	Disposal must be made according to official regulations	
Section 14 – Transport Information		
Label for conveyance	Lithium Battery Label	
UN Number	UN 3171	
Transport hazard class(es)	9	
Packing group	--	
Marine pollutant	No	
UN Proper shipping name	Battery-powered vehicle or Battery-powered equipment	
ICAO/IATA	Can be shipped by air in accordance with international Civil Aviation Organization (ICAO), TI or International Air Transport Association (IATA) DGR Packing Instructions 952 appropriately.	DGR 61 th
IMDG CODE	International Maritime Dangerous Goods Code under Special Provision 188	IMDG CODE (Amdt.39-18)
ADR	European Agreement concerning the International Carriage of Dangerous Goods by Road under Special Provision 188	ADR 2019

Material Safety Data Sheet

REPORT NO.: LCS190806165ASD

RID	Regulations concerning the International Carriage of Dangerous Goods by Rail under Special Provision 188	RID 2019
------------	--	----------

May be shipped without being declared as Class 9 dangerous goods, when meet to the requirements above.

The dangerous goods regulations require that each battery design be subject to tests contained in Section 38.3 of the UN Manual of Tests and Criteria prior to being offered for transport.

Section 15- Regulatory information

Law information

- 《Dangerous Goods Regulations》
- 《Recommendation on the Transport of Dangerous Goods Model Regulations》
- 《International Maritime Dangerous Goods》
- 《Technical Instructions for the Safe Transport of Dangerous Goods》
- 《Classification and code of dangerous Goods》
- 《Consumer Product Safety Act》 (CPSA)
- 《Federal Environmental Pollution Control Act》 (FEPCA)
- 《Resource Conservation and Recovery Act》 (RCRA)
- 《European Agreement concerning the International Carriage of Dangerous》
- 《Regulations concerning the International Carriage of Dangerous》

In according with all Federal, State and local laws.

Section 16- Other Information

The information above is believed to be accurate and represents the best information currently available to us. However, concorde makes no warranty of merchantability or any other warranty, express or implied, with respect to such information, and we assume no liability resulting from its use. Users should make their own investigations to determine the suitability of the information for their particular purposes. Although reasonable precautions have been taken in the preparation of the data contained herein, it is offered solely for your information, consideration and investigation. This material safety data sheet provides guidelines for the safe handling and use of this product; it does not and cannot advise on all possible situations, therefore, your specific use of this product should be evaluated to determine if additional precautions are required.

-- End of Report --

Lithium Battery UN38.3 Test Report

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ST/SG/AC.10/11/Rev.6, 38.3/Amendment 1

Sample name : Polymer Lithium-ion Battery

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Details information of the battery and the cell built in the battery, as following:

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MSDS

MATERIAL SAFETY DATA SHEET

Prepared For : Changzhou Fengyu New Energy Technology Co.,
Ltd.
No. 35 Leshan Road, Xinbei District, Changzhou
City, Jiangsu Province, China

Prepared By : Shenzhen LCS Compliance Testing Laboratory Ltd.
Room 101, 201, Building A and Room 301, Building C,
Juji Industrial Park, Yabianxueziwei, Shajing Street,
Bao'an District, Shenzhen, Guangdong, China

Issue Date : 2021.05.24

Report Number : LCS210513105ASD

Written by: Una. Quan

Approved by: *

Material Safety Data Sheet

REPORT NO.: LCS210513105ASD

* The MSDS is prepared based on the information provided by client. The contents and formats of this MSDS are revised as per client's request.

Section 1-Chemical Product and Company Identification

Product Name	Polymer Lithium ion Battery Pack
Model	FY24-8
Trade Mark	N/A
Ratings	24V, 8000mAh, 192Wh
Weight	1590.5g
Manufacturer	Changzhou Fengyu New Energy Technology Co., Ltd.
Manufacturer address	No. 35 Leshan Road, Xinbei District, Changzhou City, Jiangsu Province, China
Emergency Telephone	+86-519-83859198
Fax	+86-519-83859098

Section 2- Composition Information

Chemical Composition	CAS No.	Weight (%)	Trade Secret
Lithium cobaltate	12190-79-3	15 - 40	*
Graphite	7782-42-5	10 - 30	*
Phosphate(1-), hexafluoro-, lithium	21324-40-3	10 - 30	*
Copper	7440-50-8	7-13	*
Aluminium	7429-90-5	5-10	*
Nickel	7440-02-0	1-5	*

" * " The exact percentage (concentration) of composition has been withheld as a trade secret.

Section 3- Hazards Identification

Emergency overview:	N/A
Classification according to GHS	Not a dangerous substance according to GHS
Label elements:	
Hazard pictogram(s)	Not Applicable
Signal word	Not Applicable
Hazard statement(s)	Not Applicable

Material Safety Data Sheet

REPORT NO.: LCS210513105ASD

Precautionary statement(s):	
Prevention	Not Applicable
Response	Not Applicable
Disposal	Not Applicable
Environmental hazards:	No relevant information
Important symptoms:	See section 11 for more information
Section 4- First Aid Measures	
Eye contact	Flush eyes with plenty of water for least 15 minutes, occasionally lifting the upper and lower eyelids. Get medical aid.
Skin contact	Remove contaminated clothes and rinse skin with plenty of water or shower for 15 minutes. Get medical aid.
Inhalation	Remove from exposure and move to fresh air immediately. Use oxygen if available.
Ingestion	Give at least 2 glasses of milk or water. Induce vomiting unless patient is unconscious. Call a physician.
Section 5- Fire Fighting Measures	
Flash Point	N/A
Auto-Ignition Temperature	N/A
Extinguishing Media	H ₂ O, CO ₂
Special Fire-Fighting Procedures	Self-contained breathing apparatus
Unusual Fire and Explosion Hazards	Cell may vent when subjected to excessive heat-exposing battery contents
Hazardous Combustion Products	Carbon monoxide, carbon dioxide, lithium oxide fumes.
Section 6- Accidental Release Measures	
Personal precautions, protective equipment and emergency procedures: If the battery is released, remove personnel from area until fumes dissipate. Provide maximum ventilation to clear out hazardous gases. The preferred response is to leave the area and allow the vapors to dissipate. Avoid skin and eyes contact or inhalation of vapors. Remove spilled liquid with absorbent and incinerated. If leakage of the battery happens, liquid could be absorbed with sand, earth or other inert substance and contaminated area should be ventilated meantime.	
Environment precautions: Do not allow product to reach sewage system or any water source. Inform respective authorities in case of seepage into water course or sewage system. Do not allow to enter sewers/ surface or ground water.	

Material Safety Data Sheet

REPORT NO.: LCS210513105ASD

Methods and material for containment and cleaning up:

If battery casing is dismantled, small amounts of electrolyte may leak. Collect all released material in a plastic lined container. Dispose off according to the local law and rules. Avoid leached substances to get into the earth, canalization or waters.

Section 7- Handling and Storage

Handling	<p>The battery should not be opened, destroyed or incinerate, since they may leak or rupture and release to the environment the ingredients that they contain in the hermetically sealed container.</p> <p>Do not short circuit terminals, or over charge the battery, forced over-discharge, throw to fire.</p> <p>Do not crush or puncture the battery, or immerse in liquids.</p>
Storage	<p>Avoid mechanical or electrical abuse. Storage preferably in cool, dry and ventilated area, which is subject to little temperature change. Storage at high temperatures should be avoided.</p> <p>Do not place the battery near heating equipment, nor expose to direct sunlight for long periods.</p>
Other Precautions	<p>The battery may explode or cause burns, if disassembled, crushed or exposed to fire or high temperatures. Do not short or install with incorrect polarity.</p>

Section 8- Exposure Controls/Personal Protection

Engineering Controls	<p>Use local exhaust ventilation or other engineering controls to control sources of dust, mist, fumes and vapor.</p> <p>Keep away from heat and open flame. Store in a cool, dry place.</p>
Personal Protective Equipment	<p>Respiratory Protection: Not necessary under normal conditions.</p> <p>Skin and body Protection: Not necessary under normal conditions, Wear suitable protective clothing and gloves if handling an open or leaking battery.</p> <p>Hand protection: Wear suitable gloves if handling an open or leaking battery.</p> <p>Eye Protection: Not necessary under normal conditions, Wear safety glasses if handling an open or leaking battery.</p>
Other Protective Equipment	<p>Have a safety shower and eye wash fountain readily available in the immediate work area.</p>
Hygiene Measures	<p>Do not eat, drink, or smoke in work area. Maintain good housekeeping.</p>

Section 9- Physical and Chemical Properties

Form	Solid
Color	Silver
Odour	Not Applicable
pH	Not Applicable

Material Safety Data Sheet

REPORT NO.: LCS210513105ASD

Melting point/freezing point	Not Applicable
Boiling Point and Boiling range	Not Applicable
Flash Point	Not Applicable
Upper/lower flammability or explosive limits	Not Applicable
Vapor Pressure	Not Applicable
Vapor Density	Not Applicable
Relative density	Not Applicable
Solubility in Water	Not Applicable
Auto-ignition temperature	Not Applicable
Decomposition temperature	Not Applicable
Evaporation rate	Not Applicable
Flammability (soil, gas)	Not Applicable
Viscosity	Not Applicable
Section 10- Stability and reactivity	
Stability	The product is stable under conditions described Section 7
Conditions to Avoid	Heat above 70°C or incinerate. Deform, Mutilate, Crush, Disassemble, Overcharge, Short circuit, Expose over a long period to humid conditions.
Incompatible Materials	Oxidizing agents, acid, base.
Hazardous Decomposition Products	Carbon monoxide, carbon dioxide, lithium oxide fumes.
Possibility of Hazardous Reaction	Not Applicable
Section 11 – Toxicological Information	
Irritation	Risk of irritation occurs only if the cell is mechanically, thermally or electrically abused to the point of compromising the enclosure. If this occurs, irritation to the skin, eyes and respiratory tract may occur.
Sensitization	Not Applicable
Neurological Effects	Not Applicable
Teratogenicity	Not Applicable
Reproductive Toxicity	Not Applicable

Material Safety Data Sheet

REPORT NO.: LCS210513105ASD

Mutagenicity (Genetic Effects)	Not Applicable	
Toxicologically Synergistic Materials	Not Applicable	
Section 12- Ecological Information		
Ecological Toxicity	Not Applicable	
Mobility in soil	Not Applicable	
Persistence and Degradability	Not Applicable	
Bioaccumulation potential	Not Applicable	
Other Adverse Effects	Not Applicable	
Section 13- Disposal Considerations		
Product disposal recommendation	Observe local, state and federal laws and regulations.	
Uncleaned packaging recommendation	Disposal must be made according to official regulations	
Section 14 – Transport Information		
Label for conveyance	Lithium Battery Label	
UN Number	UN 3480 or UN 3171 or 3481	
Transport hazard class(es)	9	
Packing group	965	II
	952	—
	966	II
	967	—
Marine pollutant	No	
UN Proper shipping name	Lithium ion Batteries (Including lithium ion polymer batteries) Battery-powered vehicle or Battery-powered equipment Lithium ion Batteries packed with equipment (Including lithium ion polymer batteries) Lithium ion Batteries contained in equipments (Including lithium ion polymer batteries)	
ICAO/IATA	Can be shipped by air in accordance with international Civil Aviation Organization (ICAO), TI or International Air Transport Association (IATA) DGR 62 nd Packing Instructions Section IA of 965 or 952 appropriately.	

Material Safety Data Sheet

REPORT NO.: LCS210513105ASD

IMDG CODE	International Maritime Dangerous Goods Code IMDG CODE (Amdt.39-18)
ADR	European Agreement concerning the International Carriage of Dangerous Goods by Road
RID	Regulations concerning the International Carriage of Dangerous Goods by Rail
The dangerous goods regulations require that each battery design be subject to tests contained in Section 38.3 of the UN Manual of Tests and Criteria prior to being offered for transport.	

Section 15- Regulatory information

Law information


- 《Dangerous Goods Regulations》
- 《Recommendation on the Transport of Dangerous Goods Model Regulations》
- 《International Maritime Dangerous Goods》
- 《Technical Instructions for the Safe Transport of Dangerous Goods》
- 《Classification and code of dangerous Goods》
- 《Consumer Product Safety Act》 (CPSA)
- 《Federal Environmental Pollution Control Act》 (FEPCA)
- 《Resource Conservation and Recovery Act》 (RCRA)
- 《European Agreement concerning the International Carriage of Dangerous》
- 《Regulations concerning the International Carriage of Dangerous》

In according with all Federal, State and local laws.

Section 16- Other Information

The information above is believed to be accurate and represents the best information currently available to us. However, Concorde makes no warranty of merchantability or any other warranty, express or implied, with respect to such information, and we assume no liability resulting from its use. Users should make their own investigations to determine the suitability of the information for their particular purposes. Although reasonable precautions have been taken in the preparation of the data contained herein, it is offered solely for your information, consideration and investigation. This material safety data sheet provides guidelines for the safe handling and use of this product; it does not and cannot advise on all possible situations, therefore, your specific use of this product should be evaluated to determine if additional precautions are required.

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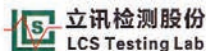Details information of the battery and the cell built in the battery, as following:
电池和电芯的详细信息见下表:

Product 产品	Cell 电芯	Battery 电池
Model 型号	FDE6360190	FY24-8
Nominal voltage 标称电压	3.7V	24V
Rated capacity 额定容量	8000mAh	8000mAh
Charge method 充电方法	2A CC(constant current) charge to 4.2V, then CV (constant voltage 4.2V) charge till charge current decline to 80mA. 以2A恒流充电至4.2V, 然后4.2V恒压充电至电流小于等于80mA。	2A CC(constant current) charge to 29.4V, then CV (constant voltage 29.4V) charge till charge current decline to 80mA. 以2A恒流充电至29.4V, 然后29.4V恒压充电至电流小于等于80mA。
Standard discharge current 标准放电电流	4A	6A
Maximum continuous charging current 最大连续充电电流	3A	3A
Maximum continuous discharge current 最大连续放电电流	15A	15A
End of discharge voltage 放电终止电压	3.0V	21V
Cell numbers 电芯个数	7PCS (7S1P)	
Dimension 尺寸	190.0*60.0*6.2(mm)	250*76.0*57.2(mm)
Weight 重量	144.8g	1590.5g

REPORT NO.: LCS210513105ASA

Page 4 of 18

Possible test case verdicts: 报告中可能用到的结论标识:	
Test case does not apply to the test object..... 测试项目不适用于该产品.....	N/A 不适用
Test item does meet the requirement 测试项目符合标准的要求.....	P(ass) 合格
Test item does not meet the requirement 测试项目不符合标准的要求.....	F(ail) 不合格
Testing: 测试:	
Date of sample received..... 样品接收日期.....	2021.05.13
Date(s) of performance of test..... 测试执行日期.....	2021.05.13 ~ 2021.05.24
Test conclusion: 检验结论: Test result: Pass 检验结果: 通过	

深圳立讯检测股份有限公司 | 地址: 广东省深圳市宝安区沙井街道衙边学子围巨基工业园A栋101、201, C栋301
 Shenzhen LCS Compliance Testing Laboratory Ltd. | Add: Room 101, 201, Building A and Room 301, Building C, Juji Industrial
 Park, Yabianxueziwei, Shajing Street, Bao' an District, Shenzhen, Guangdong, China | Tel : +(86) 0755-8259 1330 |
 Fax : +(86) 0755-8259 1332 | E-mail : webmaster@lcs-cert.com | http:// www.lcs-cert.com

Skyline

your guide in travel mobility

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Applicant : Changzhou Fengyu New Energy Technology Co., Ltd.